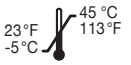
**Erkoflex** transparent/color

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#### EN: Instructions for use

**Intended use:** Thermoforming sheets are indicated for the fabrication of dental and orthodontic appliances for the treatment of all patient target groups.

**Field of application:** Erkoflex is exclusively suitable for the fabrication of thermoformed intra-oral splints like positioners, bracket transfer splints, soft splints, radiation protection splints, multi-layered protection splints.

Fabrication, see thermoforming technique brochure and videos, [www.erkodent.com](http://www.erkodent.com)  

**Contraindications:** In case of already known allergic reaction to plastics a specific allergy to Erkoflex (EVA) should be excluded.

**Precautionary measures:** Erkoflex is only recommended for the above mentioned appliances. Observe storage instructions.

**Attention:** The use is limited to the production of dental and orthodontic appliances. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Erkoflex must be reported by the user to the competent authority and to the manufacturer.

Erkoflex (approx. values): density 0.95 g/cm<sup>3</sup>, modulus of elasticity 20 MPa, elongation at break 850 %, water absorption 0.2 %, temperature resistance 54 °C

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